

Smokerlyzer[®] breath carbon monoxide (CO) monitors



CE
2797



British
Technology

Helping people to stop smoking, one breath at a time.

www.bedfont.com


bedfont[®]

Contents

| | |
|---|----|
| What is carbon monoxide (CO)? | 4 |
| What CO does to your body? | 4 |
| Why monitor CO? | 5 |
| What is a Smokerlyzer®? | 5 |
| References | 6 |
| Micro ⁺ ™ Smokerlyzer® | 7 |
| Features and benefits | 8 |
| Technical specification | 8 |
| piCO™ Smokerlyzer® | 9 |
| Features and benefits | 10 |
| Technical specification | 10 |
| piCO ^{baby} ™ Smokerlyzer® | 11 |
| Features and benefits | 12 |
| Technical specification | 12 |
| iCO™ Smokerlyzer® | 13 |
| Features and benefits | 13 |
| Technical specification | 14 |
| Consumables for piCO™, piCO ^{baby} ™, Micro ⁺ ™ | 15 |
| COdata ⁺ ™ | 15 |

What is carbon monoxide (CO)?

Carbon monoxide (CO), known as the silent killer, is a colourless, odourless, highly poisonous gas that is produced when fuels are burned incompletely. It is produced in car exhaust fumes, faulty gas boilers and tobacco smoke¹.

Oxygen is carried around the body by red blood cells and when you inhale smoke from a cigarette, the CO is absorbed into your blood through the lungs because CO binds with haemoglobin 200 times more readily than oxygen².

The CO binds with haemoglobin in the red blood cells to form carboxyhaemoglobin (COHb), preventing red blood cells from carrying oxygen².

A pack a day smoker can have a 3% to 6% COHb level in the blood, two packs a day, 6% to 10%, and three packs a day, as much as 20%³.

What CO does to your body?

Heart:

To compensate for the shortage of oxygen, the heart has to work harder (beat faster) to get enough oxygen to all parts of the body. The heart itself receives less oxygen, increasing the risk of heart damage⁴.

Circulation:

COHb causes the blood to thicken and arteries become coated with a thick fatty substance. This causes circulation problems and high blood pressure, with increased risk of a heart attack and stroke.

Breathing:

The reduced supply of oxygen means you can easily become out of breath when exercising, because there is little oxygen available for the increased demand. A lack of oxygen can also cause tiredness and a lack of concentration⁶.

Pregnancy:

Oxygen is required by a foetus for healthy growth, but the supply of vital oxygen is reduced when the mother smokes. This increases the risk of low birthweight, birth defects and even Sudden Infant Death Syndrome. Clinical studies have established a direct link between an expectant mother's breath CO level and the amount of COHb in their unborn baby's blood⁷.

Why monitor CO?

CO testing is a quick, non-invasive and cost-effective means of validating the smoking status of a significant number of clients⁸.

Smoking is one of the most preventable causes of premature death in the world⁹.

Tobacco kills up to half its users, which is more than 8 million people a year, including around 1.2 million non-smokers who have been exposed to second-hand smoke¹⁰.

"Carbon monoxide monitors offer tobacco treatment specialists an independent clinical tool which provides valuable evidence in identifying, educating, assessing and treating tobacco-dependent patients¹¹".


Monitoring patients' CO levels helps to work out their level of nicotine dependence; the more they smoke, the higher their reading will be, indicating a higher dependence on nicotine.

What is a Smokerlyzer®?

The Smokerlyzer® range consists of breath CO monitors, which measure the amount of CO in exhaled breath to help people quit smoking. The more you smoke, the higher your CO reading will be.

The reading is given in ppm, which is the number of CO molecules in a million parts of air. The ppm reading can also tell you how much CO is in the blood (the COHb reading). %COHb is the percentage of red blood cells carrying CO instead of oxygen.

The amount of CO in an unborn baby's blood is referred to as %FCOHb (percentage of foetal COHb). Some of the Smokerlyzer® range can analyse %FCOHb as an incentive to stop smoking and comply with NICE guidelines for smoking in pregnancy.

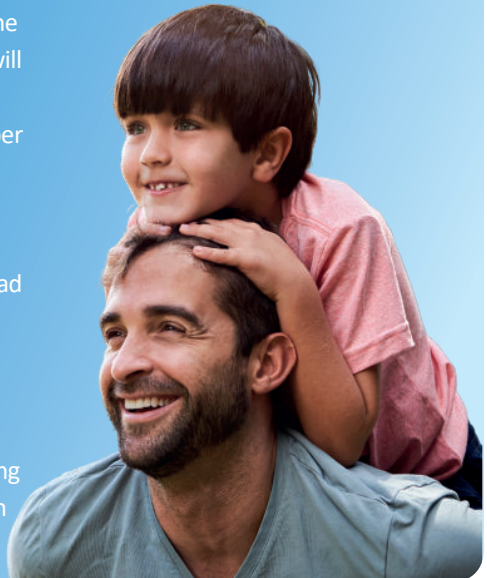

References

1. Andrew H. Carbon Monoxide 'The Silent Killer' Feasibility Study 2009/2010. 1st ed. London: London Ambulance; 2017
2. Blumenthal I. Carbon monoxide poisoning [Internet]. PubMed Central (PMC). 2017 [cited 14 March 2017]. Available from: <https://www.ncbi.nlm.nih.gov/pmc/articles/PMC1281520/>
3. How Carbon Monoxide Hurts Smokers [Internet]. Verywell Mind. 2019 [cited 21 August 2019]. Available from: <https://www.verywellmind.com/carbonmonoxide-in-cigarette-smoke-2824730>
4. Deanfield J, Shea M, Wilson R, Horlock P, De Landsheere C, Selwyn A. Direct effects of smoking on the heart: Silent ischemic disturbances of coronary flow. 2017.
5. Symptoms and Causes of Poor Circulation [Internet]. Healthline. 2017 [cited 14 March 2017]. Available from: <http://www.healthline.com/health/poor-circulation-symptoms-causes?m=2#Overview1>
6. Goldsmith, J. R. (1974) 'Carbon Monoxide and Smoking', The Lancet, 303(7868), p. 1232. doi:10.1016/s0140-6736(74)91048-4.
7. Gomez, C., Berlin, I., Marquis, P. and Delcroix, M. (2005) 'Expired air carbon monoxide concentration in mothers and their spouses above 5 ppm is associated with decreased foetal growth', Preventive Medicine, 40(1), pp. 10–15.
8. Croghan, E. (2011) 'Local stop smoking services: Service delivery and monitoring guidance 2011/2012', PsycEXTRA Dataset.
9. Health Effects [Tobacco Atlas [Internet]. Tobacco Atlas. 2018 [cited 21 August 2019]. Available from: <http://tobaccoatlas.org/topic/health-effects/>
10. Tobacco [Internet]. Who. int. 2019 [cited 21 August 2019]. Available from <https://www.who.int/news-room/fact-sheets/detail/tobacco>
11. Bittoun R. Carbon Monoxide Meter: The Essential Clinical Tool — the 'Stethoscope' — of Smoking Cessation. Journal of Smoking Cessation. 2008;3(2):69-70.
12. Health Protection Agency (HPA). Porton Down, Report No. 43/06. Pp. 10-11
13. Public Health England. An Evaluation of Filtration Efficiencies Against Bacterial and Viral Aerosol Challenges. London: Public Health England; 2017.



Micro⁺ Smokerlyzer[®]


One CO monitor for all your smoking cessation needs.



FREE
COdata⁺™
software

**Ideal for research, clinical trials and in-depth
stop smoking advice delivered by:**

- Stop smoking services
- General practices
- Schools
- Pharmacies
- Health Visitors
- Midwives
- Researchers

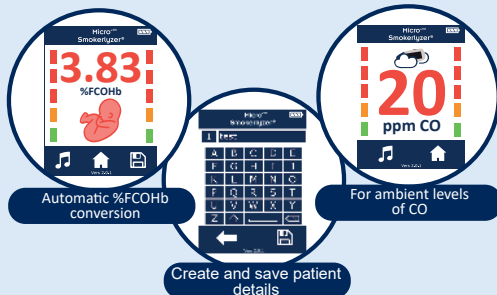

Features and benefits

- Automatic conversion to %FCOHb
- Easy to use interface
- Create and store patient details
- Large touchscreen
- SteriTouch® technology for improved infection control
- Familiar green, amber and red traffic light system, making CO levels instantly identifiable to patients
- Ambient CO monitoring
- Uses D-piece™ to filter out 99% of airborne bacteria¹³
- Single-use SteriBreath™ mouthpieces for excellent, low cost infection control
- PC interactive with COdata+™ software, free with the Micro+™ Smokerlyzer®

Technical specification

| | |
|--------------------------------------|---|
| Concentration range | 0-500ppm |
| Display | Full colour touchscreen |
| Detection Principle | Electrochemical sensor |
| Repeatability | ≤±5% |
| Accuracy | ≤±2ppm /5% * |
| Power | 3 x AA (LR6 or equivalent) - up to 1000 minutes 1 x CR2032 Lithium coin cell |
| T ₉₀ response time | <30 seconds |
| Operating temperature | 15-40°C |
| Storage/transport temperature | 0-50°C |
| Operating/storage/transport pressure | Atmospheric ±10% |
| Operating humidity | 15-90% (non-condensing) |
| Storage/transport humidity | 0-95% (non-condensing) |
| Sensor operating life | 5 years |
| Sensor sensitivity | 1ppm |
| Sensor drift | <5% per annum |
| Dimensions | Approx. 37x 77 x 140mm |
| Weight | Approx. 215g (including batteries) |
| Materials | Case: Polycarbonate/ABS blend with SteriTouch® anti-microbial additive OneBreath™: Polypropylene D-piece™: Polypropylene SteriBreath™ Eco: Paper |
| H ₂ cross interference | ≤8% |

*Whichever is greater

picoTM Smokerlyzer[®]

CO monitoring made easy.




FREE
COdata⁺TM
software

**Ideal for in-depth stop
smoking advice delivered by:**

- Stop smoking services
- General practice
- Schools
- Pharmacies
- Health visitors

As easy as:

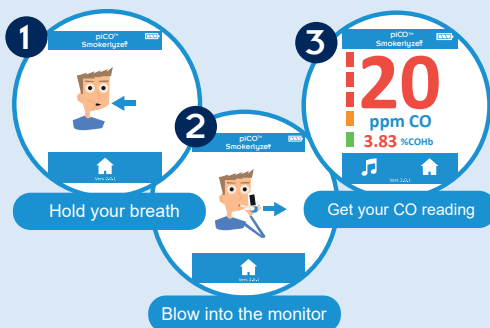

Features and benefits

- Easy to use interface
- Large touchscreen
- SteriTouch® technology for improved infection control
- Familiar green, amber and red traffic light system, making CO levels instantly identifiable to patients
- Uses D-pieces™ to filter out 99% of airborne bacteria¹³
- Single-use SteriBreath™ mouthpieces for excellent, low cost infection control
- PC interactive with COdata⁺™ software, free with the piCO™ Smokerlyzer®

Technical specification

| | |
|--------------------------------------|---|
| Concentration range | 0-150ppm |
| Display | Full colour touchscreen |
| Detection Principle | Electrochemical sensor |
| Repeatability | ≤±5% |
| Accuracy | ≤±2ppm /5% * |
| Power | 3 x AA (LR6 or equivalent) - up to 1000 minutes 1 x CR2032 Lithium coin cell |
| T ₉₀ response time | <30 seconds |
| Operating temperature | 15-40°C |
| Storage/transport temperature | 0-50°C |
| Operating/storage/transport pressure | Atmospheric ±10% |
| Operating humidity | 15-90% (non-condensing) |
| Storage/transport humidity | 0-95% (non-condensing) |
| Sensor operating life | 5 years |
| Sensor sensitivity | 1ppm |
| Sensor drift | <5% per annum |
| Dimensions | Approx. 37x 77 x 140mm |
| Weight | Approx. 215g (including batteries) |
| Materials | Case: Polycarbonate/ABS blend with SteriTouch® anti-microbial additive OneBreath™: Polypropylene D-piece™: Polypropylene SteriBreath™ Eco: Paper |
| H ₂ cross interference | ≤8% |

*Whichever is greater

piCO^{baby}™ Smokerlyzer®

CO monitoring for maternity made easy.




FREE
COdata[™]
software

Ideal for in-depth stop smoking advice delivered by:

- Midwives
- Stop smoking services
- General practice
- Schools
- Pharmacies
- Health visitors

As easy as:

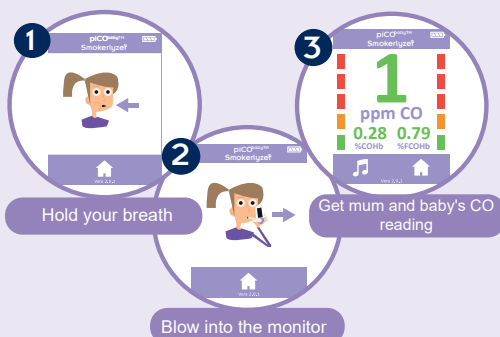

Features and benefits

- Easy to use interface
- Large touchscreen
- Automatic conversion to %FCOHb
- Familiar green, amber and red traffic light system, making CO levels instantly identifiable to patients
- Uses D-pieces™ to filter out 99% of airborne bacteria¹³
- Single-use SteriBreath™ mouthpieces for excellent, low cost infection control
- PC interactive with COdata⁺™ software, free with the piCObaby™ Smokerlyzer®

Technical Specification

| | |
|--------------------------------------|--|
| Concentration range | 0-150ppm |
| Display | Full colour touchscreen |
| Detection Principle | Electrochemical sensor |
| Repeatability | ≤±5% |
| Accuracy | ≤±2ppm /5% * |
| Power | 3 x AA (LR6 or equivalent) - up to 1000 minutes 1 x CR2032 Lithium coin cell |
| T ₉₀ response time | <30 seconds |
| Operating temperature | 15-40°C |
| Storage/transport temperature | 0-50°C |
| Operating/storage/transport pressure | Atmospheric ±10% |
| Operating humidity | 15-90% (non-condensing) |
| Storage/transport humidity | 0-95% (non-condensing) |
| Sensor operating life | 5 years |
| Sensor sensitivity | 1ppm |
| Sensor drift | <5% per annum |
| Dimensions | Approx. 37x 77 x 140mm |
| Weight | Approx. 215g (including batteries) |
| Materials | Case: Polycarbonate/ABS blend with SteriTouch® anti-microbial additive OneBreath™: Polypropylene D-piece™: Polypropylene SteriBreath™ Eco: Paper |
| H ₂ cross interference | ≤8% |

*Whichever is greater

iCOquit[®] Smokerlyzer[®]

Personal CO monitor, to help patients quit smoking remotely.

Portable, pocket-size, and with instant results, the iCOquit[®] is ideal for remote CO monitoring.



CE
2797



Visual motivation to help you stop smoking.



Ideal for remote CO monitoring in pregnancy.



Ideal for remote CO monitoring in pregnancy.

Your
Personal
CO
Monitor

Features and Benefits

- Connects to a smartphone via Bluetooth[®]
- Monitor your quit smoking progress
- Quick breath test, instant results
- Patients can easily share their progress with Health Care Professionals
- Use anytime, anywhere
- Become smoke-free

The iCOquit[®] is also ideal for:

- Health professionals
- Pharmaceutical companies
- App Developers

Technical specification


| | |
|--------------------------------------|------------------------------------|
| Concentration range | 0-150ppm |
| Detection principle | Electrochemical sensor |
| Repetability | <±5% |
| Accuracy | ≤±3ppm/15%* |
| T ₉₀ response time | <30 seconds |
| Operating temperature | 15-35°C |
| Storage/transport temperature | 0-40°C |
| Operating/storage/transport pressure | Atmospheric ±10% |
| Operating humidity | 15-90% HR (non-condensing) |
| Storage/transport humidity | 0-95% HR (non-condensing) |
| Sensor operating life | Approx. 200 tests /3 years** |
| Sensor sensitivity | 1ppm |
| Sensor drift | <5% per annum |
| Dimensions | Approx. 28mm diameter x 115mm long |
| Weight | Approx. 35g |
| Materials | Polycarbonate/ ABS blend |
| H ₂ cross interference | ≤8% |

*Whichever is greater

**Whichever comes first

iCOquit® Smokerlyzer®

For use with the iCOquit® app

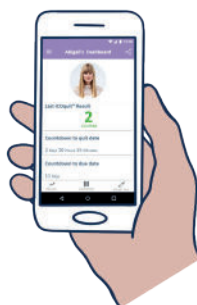

iCOquit® Smokerlyzer®

For use with the iCOquit® Mums
to Be app

Consumables for piCO™, piCO^{baby}™ and Micro⁺™

The SteriBreath™ Eco mouthpieces come in boxes of 200 mouthpieces, and each one is individually wrapped to ensure optimum infection control. The SteriBreath™ Eco mouthpiece is entirely made from paper and therefore it is 100% recyclable and 100% biodegradable, including its packaging. Even better, all the materials are from sustainable sources.



Order code: STERIBREATH-ECO (200 per box)

D-piece™: The D-piece™ is used to attach a SteriBreath™ mouthpiece to the monitor. The D-piece™ incorporates a one-way valve and an infectioncontrol filter, which are proven to remove and trap >99% of airborne bacteria¹³. The D-piece™ should be changed every four weeks, an automatic reminder will appear on the screen every 28 days.



Order code: D-PIECE-3 (12 per box)

OneBreath™ mouthpieces: The OneBreath™ mouthpiece is a single-use bacterial filter mouthpiece and can be attached directly to the monitor to take a breath sample. The OneBreath™ incorporates a one-way valve and an infection control filter, which are proven to remove and trap <99% of airborne bacteria¹³.



Order code: ONEBREATH-MP (250 per box)

COdata⁺™

COdata⁺™ is for use with the piCO™, piCO^{baby}™ and Micro⁺™ Smokerlyzer®. It is a breath carbon monoxide (CO) monitoring software with a fully integrated patient database, enabling clinicians to keep up-to-date records on patients who have given breath CO samples. COdata⁺™ can be connected to your monitor using a USB cable and allows you to print reports, save PDF reports, email results and export data to Microsoft Excel.



We also provide a range of calibration kits and replacement canisters at very competitive prices - please call for more information.





Contact Bedfont® or one of our worldwide **Smokerlyzer®** distributors for a free demonstration.

www.bedfont.com
Tel: +44 (0)1622 851122
Email: ask@bedfont.com

Our family, innovating health, for yours.

Visit www.bedfont.com/resources to view this document in other languages.



Bedfont® Scientific Ltd.
Station Road, Harrietsham, Maidstone,
Kent, ME17 1JA England
Tel: +44 (0)1622 851122 Fax: +44 (0)1622 854860
Email: ask@bedfont.com Web: www.bedfont.com



Stephen Rowe
Cristimar E4-1
Ave Juan Carlos I
Los Cristianos, Arona, 38650
Santa Cruz de Tenerife, Spain

© Bedfont® Scientific Limited 2022

Issue 22 - July 2022, Part No: MKT261
Bedfont® Scientific Limited reserves the right to change or update his literature without prior notice.
Registered in: England and Wales. Registered No: 1289798

