

RECLINING ERGOMETER

ergoselect 12



reddot award
winner



ergoline
MOVING TO HEALTH

ergoselect 12

Trailblazing functionality and expressive design, combined to create the perfect stress echo reclining ergometer. The low couch height specifically enables convenient access for elderly patients – without the interference of the ergometry unit, support elements and seat.

The extremely sturdy design supports patient weights of up to 200 kg without vibration during ergometric testing.



- patient weight up to 200 kg
- horizontal up to 38° and lateral inclination up to 39°
- remote motor control (wireless option)
- ergometry unit folds away easily
- whisper-quiet belt drive
- load range from 6 to 450 Watts
- maximum comfort
- choice of upholstery colors available
- universal interfaces
- wide range of expansion options



reddot award
winner



Max. patient weight up to 200 kg

The stable base frame and the vibration-free design guarantee the highest level of safety and enable the use of the reclining ergometer with morbidly obese patients.



Hip support

The variably adjustable hip pad ensures secure lateral patient support during stress echocardiography examinations in lateral positions.



Underarm support

Together with an integrated handle, the easily adjustable underarm pad offers additional patient support during the lateral tilting of the stress echo reclining ergometer.



Headrest


The comfortable headrest with lateral support can be moved over a wide range for easy adjustment to the patient's body size.

The laterally attached speed display is easy to read in any position and can be folded away when not in use.



Drop section for echocardiography

The easily folding cushion element considerably simplifies ultrasound examinations in lateral patient positions.

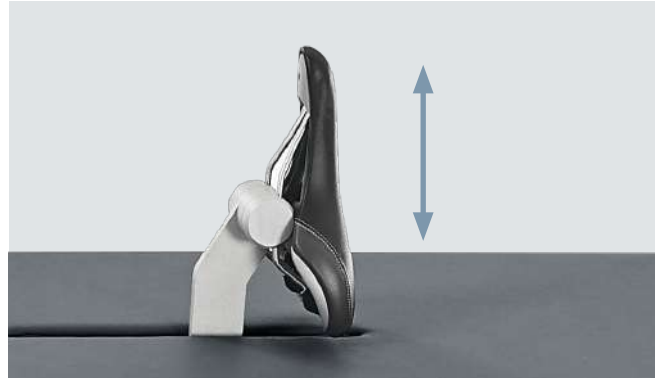

Examination couch

A few hand movements turn the ergoselect 12 into a full-fledged comfortable examination couch – the ultralight and compact ergometry unit is simply unlocked and lowered until flush with the couch surface. After removing the seat and shifting or removing the lateral support elements, no inconvenient rails or mounting elements remain in the couch surface.



Pedal shoes


The specially developed pedal shoes secure the patient's feet at the ergometry unit and reliably prevent slipping off the pedal cranks, even in lateral position.



Speed display (cadence)

During the ergometric test, the patient display shows the current speed for control, while additional icons warn against overly low or high speed.

The display is directly connected to the headrest and can be folded away laterally when not in use.

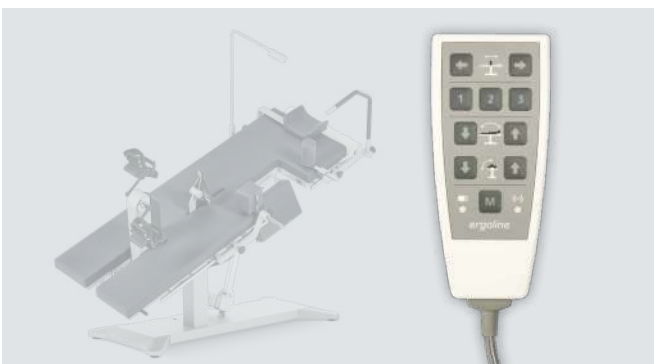

Remote control

The examination position and seat settings can be conveniently adjusted via remote control with multi-position memory. A wireless version is available on request. The independent pivot areas for both motors make it easy to move the couch into any position.



Versatile functions


ergoselect 12 is not only a perfect stress echo reclining ergometer, but can also easily be used for other applications.





Emergency lowering

In the event of a power failure, the built-in battery-driven emergency lowering feature ensures that the ergometer is moved to the starting position so the patient can safely get off.

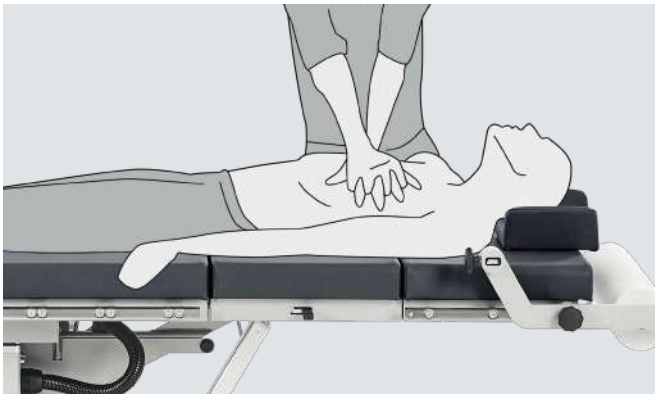

Chest compressions

The completely vibration-free design of the reclining ergometer even allows for performing chest compressions in horizontal position directly on the couch surface without the need to move the patient.



Hygienic upholstery fabric in a variety of colors

The high-quality upholstery fabric with its fine microstructure is an ideal choice:

It is durable, easy to disinfect, and antibacterial, to name just a few of the outstanding material properties. The pad cushions are available in many colors to match any practice or hospital interior design.



ECG connection/interfaces

All leading ECG manufacturers support a control option for ergoline ergometers in their exercise ECG or ergospirometry systems.

ergoline ergometers offer a wide range of connection options. In addition to the standard RS-232 and USB interface, wireless connections via Bluetooth or WiFi are available as well.

ergoselect 12 – modules, expansion options



Automatic blood pressure measurement

The optionally integrated, automatic blood pressure measurement works with a specifically developed cuff microphone. The combination of auscultatory measurement and sophisticated algorithms for artifact suppression yields precise measurement values – even at high loads.



Integrated SpO2 measurement

For precise monitoring of the patient's oxygen saturation during the stress test, the reclining ergometers can be equipped with an SpO2 measurement module (control terminal T).

A wide selection of suitable SpO2 sensors is available in different sizes.





Control terminal, on separate stand

Control terminals on a stand are available for the easy performance of custom ergometric test protocols or manual load adjustment.



Wireless remote control

The motors of the reclining ergometers can be conveniently adjusted without the hassle of cables.



Lap belt

A laterally mounted belt ensures additional safety during the examination – especially for morbidly obese patients.



Stainless steel arm rest

A universally adjustable board can be attached laterally for performing heart catheter examinations.



Additional handgrip

A sturdy handgrip can be attached at the right side.



IV pole

An IV pole can be easily attached to the DIN rails.





PREMIUM QUALITY



MADE IN GERMANY

ergoline GmbH

Lindenstraße 5 | D-72475 Bitz, Germany

Phone: +49-(0)-7431-9894-0 | Fax: +49-(0)-7431-9894-128

email: info@ergoline.com | www.ergoline.com